


3 Channel (5 lead) Electrode Placement (1st option)

Five color-coded leadwires are used to create a 3-channel ECG recording. This is a typical electrode placement. Refer to your Analysis System software and the physician for a recommended position.

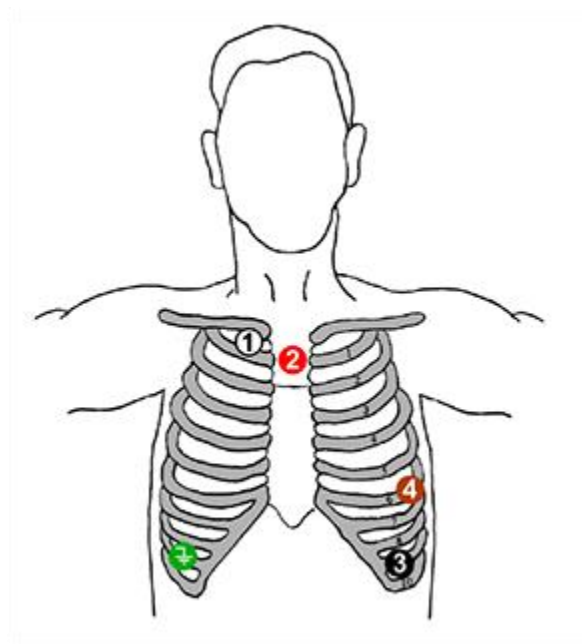

#	Channel	Color	Location
1	3-	White	Next to the right Manubrium border on the Clavicle
2	1-, 2-	Red	Centered on the Manubrium
3	2+, 3+	Black	Lower left rib margin over bone
4	1+	Brown	Left Anterior Auxiliary line on the 6th rib
5	⊕	Green	Lower right rib margin over bone